## FR

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CONSERVER ENTRE 4°C & 22°C

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## EN

**CICAVIT DM+ CICATRICES** is a medical device indicated for the treatment of scars. The silicone formula maintains a moist healing environment. Immediately soothes and relieves itching and reduces redness and tightness. Helps reduce and prevent red, thick and rigid scars forming. Flattens and smoothes scars, making them less visible.

Soft, non-oily and non-sticky transparent texture that spreads easily, forming a semi-occlusive, filmogenic, invisible film for barrier protection.

## INDICATIONS

CICAVIT DM + Cicatrices is indicated for the treatment of localized or extensive scars resulting from post-dermatological or surgical procedures skin lesions (closed wounds) or superficial burns (laser, radiotherapy). CICAVIT DM + Cicatrices can be applied to all areas of the body, including the face, joints and mobile areas. Adults.

## INSTRUCTIONS FOR USE

Apply twice a day to closed wounds and scars, on perfectly clean and dry skin. Do not use more than 30 days.

## WARNINGS AND PRECAUTIONS CONCERNING USE

CICAVIT DM + Cicatrices is for external use only. If the problem persists, stop use and consult a physician. This product should be kept out of the reach of children. It should not be used after the expiration date printed on the tube. Do not apply to mucous membranes or in the eyes. Keep for 12 months once opened. Keep at an ambient temperature.

## SIDE EFFECTS

As with any product applied to the skin, allergic skin reactions and irritation may affect some patients.

## CONTRAINDICATIONS

Do not use if allergic to any of the components. Do not use on open wounds. Do not use at the same time as a topical medicament. In the case of a skin disorder, consult your physician. The product is not indicated for women who are pregnant or breast-feeding.

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