## NEAUVIA INTENSE ROSE

HYALURONIC ACID HYDROGEL

DESCRIPTION

Neauvia Intense Rose is a sterile, non-pyrogenic, injectable, monophasic hydrogel.

The product consists of biophysically purified sodium hyaluronate of animal origin, derived from bacterial fermentation, which is non-pathogenic for humans. The hydrogel is gelatinous, transparent and colorless. The dispositif is delivered in a sterilized, single-use syringes with a luer-lock connection. The package contains the device, instruction leaflet, needle and product identification labels.

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INDICATIONS

Neauvia Intense Rose is an injectable medical device indicated for the correction of the atrophy of the vulva and the symptoms related to it via intradermal injection.

The product degrades slowly after implantation.

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**PREPARATION FOR TREATMENT**  
-Use sterile gloves - 21G up to 25G.

-Collect data on the patient's medical history (current infections, drug use, ongoing dental treatment, surgery, allergies, etc.).

-Inform the patient about the indications, contraindications and obtain the patient's signature on the informed consent form.

**ANESTHESIA**  
In order to reduce any possible reaction, Neauvia Intense Rose does not contain local anaesthetics.

It is advisable using a local nerve block technique (truncular).

Check whether to use local anaesthetics. In this case, it's advisable using a local nerve block technique (truncular).

Procedure for treatment: Inject the skin thoroughly with iodopovidone or chlorhexidine.

-Syringe with a syringe protection lid.

-Screw the needle firmly onto the luer-lock connection of the syringe.

-Wait until the needle is sharp.

If the needle is obstructed, do not increase pressure on the plunger.

-Inject slowly using the retrograde sliding injection technique.

-During injection, the product should be shaped with the operator's fingers.

-Massage the treated area gently to "distribute" the product.

**POST-TREATMENT ADVICES**  
An anti-inflammatory drug may be administered to reduce any redness or swelling that may appear after the first application.

It can be applied to the treated area, although not in direct contact with the skin, to reduce the inflammatory effects typically associated with the injection.

After 24 hours after treatment, do not apply cosmetics to the treated area.

**PROCEDURE FOR THE TREATMENT**

-Disinfect the skin thoroughly with iodopovidone or chlorhexidine.

-Screw the needle firmly onto the luer-lock connection of the syringe.

-Press gently on the plunger in order to expel residual air from the syringe.

If the needle is obstructed, do not increase pressure on the plunger.

-Inject slowly using the retrograde sliding injection technique.

-During injection, the product should be shaped with the operator's fingers.

-Massage the treated area gently to "distribute" the product.

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