**FR** NCTF®  
135 / 135 HA

**EN** NCTF®  
135 / 135 HA

**ES** NCTF®  
135 / 135 HA

**DE** NCTF®  
135 / 135 HA

**IT** NCTF®  
135 / 135 HA

**NL** NCTF®  
135 / 135 HA

**PT** NCTF®  
135 / 135 HA

**EN** NCTF®  
135 / 135 HA

**NCTF® 135**  
Acide hyaluronique  
+ solution polyvalentisante

**NCTF® 135HA**  
Acide hyaluronique hautement concentré  
+ solution polyvalentisante

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#### 5 Acides nucléiques

Deoxyadenosine  
Deoxycytidine  
Deoxyguanosine  
Deoxythymidine  
5-Méthyl-2'-deoxycytidine

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Do not inject into blood vessels.

No use in:

• patients with a skin alteration or disease of any kind in the treated area.

• patients with a history of auto-immune disease or under immunosuppressive therapy.

• patients with hypersensitivity or a known allergy to any of the components.

• pregnant or breast-feeding women.

• children.

#### PRECAUTIONS FOR USE

There are no clinical data available in terms of tolerance with regard to the injection of NCTF® 135 and NCTF® 135HA in an area that has already been treated with another "permanent" filling agent.

NCTF® 135 and NCTF® 135HA are not injected at the same level.

Patients receiving an anticoagulant treatment or a platelet aggregation inhibitor (for example, aspirin) must be informed of the increased risk of bruising and mild bleeding at the injection site.

The patient should be given the recommendation not to use makeup, for 12 hours following the injection and to avoid sunbathing or exposure to the sun, UV rays, freezing temperatures as well as use of saunas or steam rooms for at least 48 hours after the injection.

Training in mesotherapy techniques is recommended for the use of NCTF® 135 and NCTF® 135HA.

Application as a medium or deep chemical peel or dermabrasion.

NCTF® 135 and NCTF® 135HA do not need to be injected at the same time.

A local anesthetic patch (cream) can be applied prior to the injection.

No use of topical anesthetic products can cause local redness or hypersensitivity.

No use of NCTF® 135 and NCTF® 135HA in beauty spots (naevi) and in active acne lesions.

#### INCOMPATIBILITIES

There is a known incompatibility between hyaluronic acid and temporary ammonia salts such as benzalkonium chloride.

NCTF® 135 and NCTF® 135HA should therefore never come into contact with such products or with medical/surgical equipment treated with this type of product.

#### UNDESIRABLE EFFECTS

The practitioner must inform the patient that there are possible undesirable effects related to the implantation of this device, such as:

Redness or mild local inflammation that usually disappears within 24 to 48 hours.

Mild oedema and small haematomas may occur, but disappear within 48 hours.

Minimal bleeding at the injection site that stops rapidly and spontaneously.

The patient must notify the practitioner as soon as possible of any inflammatory reactions lasting more than one week or the appearance of any other adverse effect.

The practitioner must treat the adverse effect with suitable treatment.

Any other adverse effect related to the injection of NCTF® 135 and NCTF® 135HA must be reported to the distributor and/or manufacturer.

#### INSTRUCTIONS FOR USE

Before starting treatment, the patient must be informed of the indications of the device, its contraindications, its incompatibilities and its possible undesirable effects.

All asepsis and hygiene precautions must be taken to prepare the site, the material and the patient prior to the injection.

1/ Connect a 180 needle to a 3 ml syringe (provided in the kit) and aseptically withdraw NCTF® 135 or NCTF® 135HA solution.

2/ Disconnect the 180 needle and connect the suitable sterile needle (provided in the kit) to the syringe.

3/ Remove the cap of the needle and inject the NCTF® 135 or NCTF® 135HA into the skin.

NCTF® 135 and NCTF® 135HA can also be used with mesotherapy devices (gun or roller).

Data relating to the injection (patient details and product batch number) must be kept by the practitioner.

#### PROTOCOL

Standard protocol of 5 treatments, the first 3 at intervals of 2 weeks, and the last 2 at an interval of 1 month, once per 2 weeks.

Additional treatments can be given every 3 or 6 months if the practitioner deems it necessary with regard to skin quality.

#### WARNINGS

• Check the expiry date on the label.

• Do not reuse.

• After use, the needles must be thrown away in a suitable safe container.

• The use of the product involves risks (e.g. cross contamination) for the patient.

• Do not use if the packaging is open or damaged.

#### STORAGE / CONSERVATION CONDITIONS

Store NCTF® 135 and NCTF® 135HA between 2 and 25°C (36-77°F) in a dry place protected from light and moisture. Handle with caution.

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NCTF® 135HA: Anti-aging mesotherapy product indicated for intensive revitalization and hydration of tired or dull skin, treatment of wrinkles and redensification of mature or slack skin.

NCTF® 135 and NCTF® 135HA are indicated to be injected in the epidermis or dermis in the following areas:

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