concerned about TSS.

YOUR EASY-PEASY GUIDE TO INSERTING AN APPLICATOR TAMPON:

1. Relax! Remove the wrapper.
2. Pull out the smaller inner tube until it's fully extended. Make sure the string hangs out the small tube.
3. Hold the base of the outer tube and gently insert the applicator tip into the vaginal opening until your fingers touch your body.
4. While still holding the applicator outer tube, push the inner tube completely into the outer tube until the bottoms of the tubes meet.
5. Pull out the applicator. The string should hang outside your body.

NON-APPLICATOR BABES:

1. Relax! Unwrap from the seam on the middle of the wrapper.
2. Hold the tampon with your thumb and pointer finger and use your other hand to pull out the string.
3. With the pointer finger of your dominant hand, push tampon into your vagina. Insert it until you reach the base of your finger. The string should hang outside your body.

REMOVING A TAMPON:

1. Relax! Gently pull the string so the tampon slides out.
2. You can use your fingers if you can't find the string.
3. Wrap up & bin the tampon - don't flush!

TLET'S TALK TOXIC SHOCK

TOXIC SHOCK SYNDROME (TSS) is a rare but serious illness that can be fatal. TSS can occur in men, women & children and is caused by toxins produced by the bacterium *Staphylococcus aureus*. The estimated incidence of TSS is 1 to 17 per 100,000 menstruating women. Almost 50% of TSS cases occur in menstruating women, especially women under 30. Menstrual TSS is associated with tampon use. You can reduce your risk of getting TSS by alternating tampon use with pads and avoid the risk of menstrual TSS by not using tampons. Early recognition & treatment of TSS symptoms are important.

SYMPTOMS:

TSS can occur during or just after menstruation, and symptoms can rapidly progress from flu-like to a serious & potentially fatal illness. The following symptoms below may not all occur at the same time:

- Fainting, dizziness, weakness
- Sudden high fever over 39 degrees C
- Muscle aches, headaches, sore throat
- Sunburn-like rash
- Diarrhoea and/or vomiting

If you get any of these symptoms, you must immediately remove your tampon, get medical attention and tell your doctor that you are menstruating and using tampons and

If you had TSS in the past, you can get it again, with or without further tampon use. Talk to your doctor before using tampons if you have had TSS or symptoms of TSS in the past.

WHAT'S IN A TAMPON?

100% organic cotton certified by the Global Organic Textile Standard & produced according to the International Labour Organisation's workers' standards.

WHEN TO CHANGE A TAMPON:

Change your tampon as often as needed but never leave it in longer than 8 hours. Remove the current tampon before inserting another & always remember to remove the last tampon at the end of your period.

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