#### EN All-in-one solution with hyaluronate

Cleaning, removal of protein deposits, disinfecting, rinsing, storing, inserting and moistening. For soft contact lenses.

#### Recommended for sensitive eyes

Contains hyaluronate that enhances the moistening of lenses and increases the tolerability of lens-wear due to its viscous-elastic properties. The solution hydrates lenses due to its 'water lock' not only after insertion but also throughout the day. It thereby aids the wearing comfort of the lens and diminishes eye irritation and dryness. The characteristics of the cornea and its surrounding tissue, in terms of its acidity, were taken into account during the development of this solution resulting in a product with corresponding specifications. Sterile manufacturing.

#### Composition:

POLYQUATERNIUM 0,00065%, PHMB 0,00002%, HYALURONATE, SODIUM CHLORIDE, EDTA, POLOXAMER, BORAX/BORIC ACID, PURIFIED WATER.

#### Precautions and instructions for use

Wash your hands carefully before removing the lenses from your eyes or the lens case. Dry your hands using a clean towel.

Do not use when tamper-evident seal is broken before the first time use. Unscrewing the cap from the bottle will break the tamper-evident seal.

Immediately after removal of the lenses, clean them with a few drops contact lens solution.

Clean the lenses by carefully rubbing them with your index finger in the palm of your other hand for 10-15 seconds.

Rinse the lenses with contact lens solution.

Fill your lens case with fresh contact lens solution.

Place the lenses in the lens case for disinfection and store the lenses for at least 6 hours.

Remove the lenses from the lens case after the disinfecting cycle.

Rinse the lenses with contact lens solution.

Insert the lenses into your eyes.

Rinse the lens case with contact lens solution and allow the lens case to dry in the air.

Repeat disinfection steps when the lenses are stored unused in the lens case for a period longer than 7 days.

Do not touch the dropper tip of the bottle as this may contaminate the solution.

Close the bottle tightly immediately after use.

Replace your lens case frequently to prevent infection due to contamination.

#### Storage conditions

Use within 3 months after opening.

Store between 4 °C and 25 °C. (See image 1)

Protect from sunlight.

Keep out of reach of children. (See image 2)

Keep the solution in its original packaging.

#### Warnings and contra-indications

Do not use the contact lens solution if you are allergic to one of the ingredients. Stop wearing contact lenses when you experience eye irritation or when you have an eye infection and consult a doctor.

If you are prescribed eye medication, consult your ophthalmologist or contact lens specialist before using the medication in combination with contact lens solution.

Do not use when the packaging is damaged. (See image 3)

Do not use after the expiry date.

Do not change your contact lens care system without consulting your contact lens specialist.

Always replace any remaining solution in the lens case with fresh solution.

Never use tap water as an alternative for contact lens solution.

Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.

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