GB

Ophthalmic solution to moisturize and lubricate the ocular surface.  
Moisturising drops with Hyaluronic acid.  
Suitable for use with or without contact lenses.  
RELIEF OF OCULAR DRYNESS.

**INTENDED USE:**

Moisturises and lubricates the eye.  
Relieves the sensation of dryness and ocular irritations caused by environmental conditions, tiredness, extended digital devices use or contact lens wear.

**CHARACTERISTICS AND MODE OF ACTION:**

Ophthalmic Hydralarme contain sodium hyaluronate, a natural polymer, which is also present in the structures of the human eye.  
The physical characteristics of Sodium Hyaluronate confer "viscoelastic" and water retention properties.  
Ophthalmic Hydralarme provides long lasting relief and maximum comfort.

**INSTRUCTIONS FOR USE:**

- Wash and dry hands thoroughly before use.
- Check if the bottle is intact to ensure sterility.
- To open bottle, turn screw cap counter-clockwise to break closure.
- Tilt head back and release two drops from inverted bottle into each eye, taking care not to touch the eye with the bottle.
- Use with contact lenses: Apply one or two drops before insertion, during wear or before removing lenses.
- Suitable for adults and children.

**PRECAUTIONS:**

- Once the bottle is open, use the product within 3 months.
- Do not use if the bottle is damaged.
- To avoid contamination, do not touch the dropper.
- Do not touch the eye with the bottle, this to avoid erosions or eye infections.
- Do not use after the expiry date that appears on the outer package.
- Store at room temperature.
- Keep out of children's reach.
- If you experience problems after use, seek medical advice.
- Do not use if you are allergic to any of the components.
- Before using ophthalmic medicine, inform your doctor you are using this product.
- Do not use while also using medication or another product that is applied on the ocular surface, to prevent altering the functions of the product.

**Composition per ml:**

2 mg Sodium hyaluronate, 0.0015 mg Polyhexanide (PHMB), 0.1 mg Sodium Edetate (EDTA), isotonic phosphate buffer, purified water.  
Last revision date: 2019/06

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**AVIZOR S.A.**